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Geistlich Pharma AG  
CH-6110 Wolhusen  
Switzerland

## ENGLISH

Geistlich Bio-Oss® is a sterile, biocompatible porous bone mineral for use in periodontal, and oral and maxillofacial surgery.

### DESCRIPTION:

Geistlich Bio-Oss® is a porous bone mineral matrix. It is produced by removal of all organic components from bovine bone. Due to its structure Geistlich Bio-Oss® is physically and chemically comparable to the mineralized matrix of human bone. It is available in spongiosa (cancellous) granules and blocks. Geistlich Bio-Oss® is sterilized by gamma -irradiation.

### PROPERTIES / ACTIONS:

The anorganic bone matrix of Geistlich Bio-Oss® has macro- and microscopic structures similar to human bone. The formation and ingrowth of new bone at the implantation site of Geistlich Bio-Oss® is favored, due to its trabecular architecture, interconnecting macro and micro pores and its consistency.

### INDICATIONS FOR USE:

Geistlich Bio-Oss® is intended for the following uses:

- > augmentation or reconstructive treatment of the alveolar ridge
- > filling of infrabony periodontal defects
- > filling of defects after root resection, apicoectomy, and cystectomy
- > filling of extraction sockets to enhance preservation of the alveolar ridge
- > elevation of the maxillary sinus floor
- > filling of periodontal defects in conjunction with products intended for Guided Tissue Regeneration (GTR) and Guided Bone Regeneration (GBR)
- > filling of peri-implant defects in conjunction with products intended for Guided Bone Regeneration (GBR).

### CONTRAINDICATIONS:

Geistlich Bio-Oss should not be used in the presence of infected wounds.

### DIRECTIONS FOR USE:

The general principles of sterile handling and patient medication must be followed when using Geistlich Bio-Oss®.

- > The filling of periodontal defects with Geistlich Bio-Oss® requires successful treatment of the periodontal lesion (root planing, debridement) prior to the implantation. The defect should be covered with a membrane (e.g., Geistlich Bio-Gide®) for optimal tissue regeneration.
- > After exposure of the bony defect with mucoperiosteal flap, all granulation tissue must be carefully removed.
- > The material should be placed in direct contact with well vascularized, bleeding bone surfaces. Cortical bone should be mechanically perforated to facilitate ingrowth of new blood vessels and bone forming cells.
- > Application:
  - a) Geistlich Bio-Oss Pen® spongiosa (cancellous) granules: Please see pictograms at the end of the page or on the package label to follow the instruction below.
    1. Holding the Geistlich Bio-Oss Pen® firmly, tear off the protective seal (A) and unscrew the protection cap (B). This exposes the applicator and filter cap.
    2. Do not remove the red/green filter cap from the applicator at this time.
    3. For complete wetting of Geistlich Bio-Oss®, hold the applicator with both hands and place the filter cap into the sterile physiological saline solution or the patient's blood. Completely retract the plunger. For 0.5 cc applicator, one retraction is sufficient to complete wetting of Geistlich Bio-Oss®. For 1.0 cc and 1.5 cc applicators, depress the plunger, using thumb pressure, to eject the wetting fluid without compressing Geistlich Bio-Oss®. Re-wet with the same wetting fluid by retracting and depressing the plunger approximately 5 times. Alternatively, use a separate syringe (not included) to wet Geistlich Bio-Oss®. Directly inject the wetting fluid of choice through the filter cap into the Geistlich Bio-Oss® granules.
    4. Once wetted, gently depress the plunger to eject any excess fluid.
    5. Remove the filter cap from the applicator (A). Attach the curved applicator tip by screwing it on to the end of the applicator (B). The applicator is now ready for use. Using thumb pressure, depress the plunger and apply Geistlich Bio-Oss® directly to the surgical site.
  - b) Geistlich Bio-Oss® spongiosa (cancellous) granules in vials: Mix Geistlich Bio-Oss® granulate with the patient's blood or with physiological saline solution before the implantation. The granulate is placed into the defect, using sterile instruments (spatula or spoon). Use of excessive force will result in crushing of particles and loss of trabecular architecture.
  - c) Geistlich Bio-Oss® spongiosa (cancellous) bone blocks: The Geistlich Bio-Oss® block may be adapted to the desired size with a scalpel and placed into the bone cavity. If necessary, the block may be moistened with the patient's blood or with physiological saline solution.

The shaped block is then placed loosely into the bone cavity in direct contact with well vascularized and bleeding bone.

- > In situ modeling may be performed with a sterile spatula or other suitable instrument.
- > Overfilling of the defects should be avoided.
- > It is advisable to cover Geistlich Bio-Oss® with a membrane barrier (e.g., Geistlich Bio-Gide®).
- > When closing the wound, the soft tissue flap should completely cover the implanted Geistlich Bio-Oss®, and should be fixed by sutures.
- > If primary wound closure cannot be achieved completely, further mobilization of the flap (incision through the periosteum) should be performed.
- > A surgical dressing may be placed over the wound for one to two weeks.
- > Sites grafted with Geistlich Bio-Oss® should be allowed to heal approximately 6 months prior to implant placement.
- > A basic requirement for successful periodontal treatment includes control of any bacterial infection as well as thorough oral hygiene. It is advised that, preceding the surgical intervention, there be a hygiene phase, which would include proper instruction for the patient. A postoperative maintenance phase can ensure long-term therapeutic success.

#### PRECAUTIONS:

Geistlich Bio-Oss® should only be used by trained dentists or oral surgeons. In order to facilitate the formation of new bone Geistlich Bio-Oss® should only be implanted in direct contact with a well vascularized bony tissue (selective osteoplasty of adjacent cortical bone may be necessary) and if necessary microfracturation of the adjacent bone might be performed. From experience, mechanical loading (compression loading) of Geistlich Bio-Oss augmented areas is possible after 6 months at the earliest. Importantly the appropriate timing of implant insertion depends on the local bone volume at the time of implant insertion. In larger defects a mixture of autogenous bone or bone marrow may improve the formation of new bone.

Geistlich Bio-Oss® should be secured to prevent motion and migration, use in areas where the graft can be adequately contained.

Do not leave the defect open.

Geistlich Bio-Oss® should be used with special caution in patients with:

- > acute or chronic infection (osteomyelitis) at the surgical site
- > metabolic diseases (diabetes, hyperparathyroidism, osteomalacia)
- > osteoporosis
- > severe renal dysfunction, severe liver disease
- > high dose therapy with corticosteroids
- > vascular impairment at the implant site
- > autoimmune disease
- > radiotherapy
- > heavy smoking.

Effect on pregnant or lactating women is not known.

#### ADVERSE REACTIONS:

Incompatibility reactions with Geistlich Bio-Oss® cannot be excluded. Possible complications which may occur with any surgery include swelling at the surgical site, flap sloughing, bleeding, local inflammation, bone loss, infection or pain.

#### STORAGE AND HANDLING:

If the sterile packaging is damaged or opened, the product must not be used. The contents of the blister package or bottle are designed for single use only. Discard any unused material after opening. Resterilization is not possible. Do not use after the expiration date. Store at controlled room temperature 15°–25 °C (59°–77 °F) and in a dry place.

#### PRESENTATION:

Re-Order No.	Product	Weight	Particle Size
20111	Geistlich Bio-Oss® spongiosa (cancellous) granules	0.25 g	0.25 – 1.0 mm
20112	Geistlich Bio-Oss® spongiosa (cancellous) granules	0.5 g	0.25 -- 1.0 mm
500304	Geistlich Bio-Oss® spongiosa (cancellous) granules	1.0 g	0.25 – 1.0 mm
20113	Geistlich Bio-Oss® spongiosa (cancellous) granules	2.0 g	0.25 – 1.0 mm
20114	Geistlich Bio-Oss® spongiosa (cancellous) granules	5.0 g	0.25 – 1.0 mm
20121	Geistlich Bio-Oss® spongiosa (cancellous) granules	0.5 g	1.0 – 2.0 mm
500305	Geistlich Bio-Oss® spongiosa (cancellous) granules	1.0 g	1.0 – 2.0 mm
20122	Geistlich Bio-Oss® spongiosa (cancellous) granules	2.0 g	1.0 – 2.0 mm

Re-Order No.	Product	Weight	Particle Size
20115	Geistlich Bio-Oss Pen® spongiosa (cancellous) granules	0.25 g	0.25 – 1.0 mm
20116	Geistlich Bio-Oss Pen® spongiosa (cancellous) granules	0.5 g	0.25 – 1.0 mm
20126	Geistlich Bio-Oss Pen® spongiosa (cancellous) granules	0.5 g	1.0 – 2.0 mm

Re-Order No.	Product	Block size (approx.)
20131	Geistlich Bio-Oss® spongiosa (cancellous) block	1 x 1 x 2 cm

**Distributed by:**

Geistlich Pharma North America Inc.  
Princeton, New Jersey 08540  
1-(855) 799-5500  
dental.geistlich-na.com

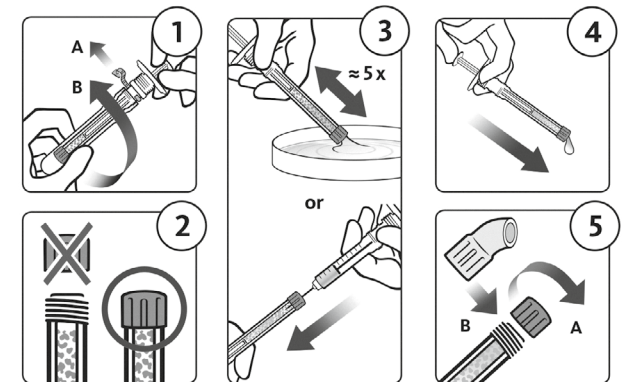
**Manufacturer:**

Geistlich Pharma AG  
CH-6110 Wolhusen  
Switzerland

Made in Switzerland

CAUTION: Federal law restricts this device to sale by or on the order of a dentist or physician.  
The product, Geistlich Bio-Oss Pen<sup>®</sup>, is marketed and distributed under the US Patent numbers  
8,491,592 and D721,803 S

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